



i mylife™ SafetyLancets

Capillary blood sampling with reduced risk of accidental needle injuries

The innovative patented design makes capillary blood sampling safe and easy. The single release function and a sealed protection cap guarantee maximum security and protect patients as well as HCP's from cross-contamination or accidental finger lancing.




Safety Features

- Safety seal ensures that the mylife™ SafetyLancet is unused
- Protective cap maintains sterility and prevents from contamination prior use
- The safely concealed needle before and after use prevents from needle stick injuries
- Inoperable after use thanks to single release function


Comfort Features

- High speed penetration minimizes pain, making blood sampling feel like a soft touch
- Superior needle quality (tri-beveled needle tip) guarantees an optimum blood flow with maximum comfort
- No pre-loading or assembly prior use required


How to use the mylife™ SafetyLancet

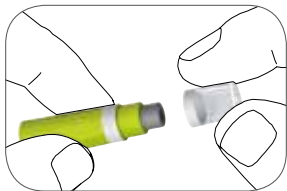


1 Simply twist and remove the protective cap from the mylife™ SafetyLancet.

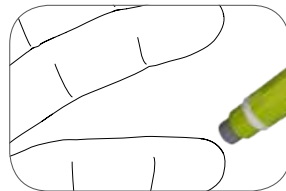

2 Place the mylife™ SafetyLancet on the finger.

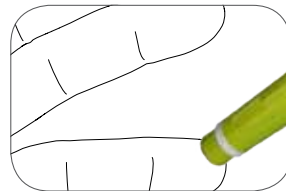

3 Push the raised platform firmly against the chosen site to activate the lancet mechanism.

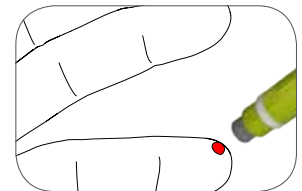

4 The needle retracts immediately after use. Thanks to a single-use release function the mylife™ SafetyLancet cannot be re-used inadvertently.



5 Dispose of the mylife™ SafetyLancet, in accordance with local regulations.





mylife™ SafetyLancets

Sterile disposable safety lancets for safe and comfortable routine blood sampling

Available in boxes of 200 pcs. with convenient slider lid



Designed for blood glucose monitoring and other diagnostic tests in

- Diabetes clinics
 - Hospitals
 - Private homecare
- etc.

Maintains sterility for 3 years

Sterilizing dose determined in accordance with ISO 11137

Sterilized by Co 60 Irradiation

Ref. No.	Qty.	Description	Gauge	Needle Diameter	Point Length	Body Outer Diameter
7100031	200 pcs	mylife™ SafetyLancets	28G	0.36 mm	3.0 +/- 0.3 mm	10.5 +/- 2.0 mm

Customer Care: 0800 092 6787

More information on www.mylife-diabetescare.co.uk

Ypsomed Ltd. // Blackwood Hall Business Park // North Duffield, Selby
North Yorkshire, YO8 5DD // PH: 0844 856 7820 // www.ypsomed.co.uk