




Sterile disposable lancets


To enable effective and comfortable capillary blood sampling

Virtually pain free

Electro-polished tri-beveled needle tip guarantees a smooth and sharp finish to obtain optimum blood samples with maximum comfort.

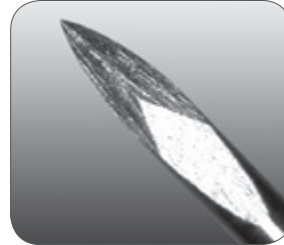


mylife™ Lancets
(150 x zoom)

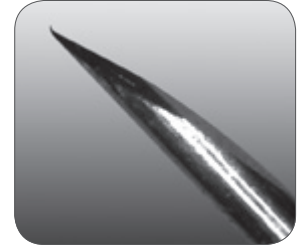

Competitors' Lancets
(150 x zoom)

Exposed Needle Length (ENL)

Needle is perfectly positioned to ensure consistent needle concentricity and exposed needle length.

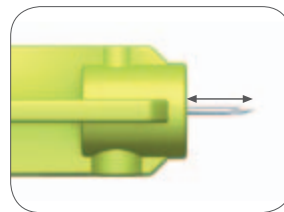

Optimized safety


No LDPE (low density polyethylene) material remains on needle tip after removing lancing cap.



Designed for use with mylife™ Softlance

And most other sampling devices

Safe disposal after use

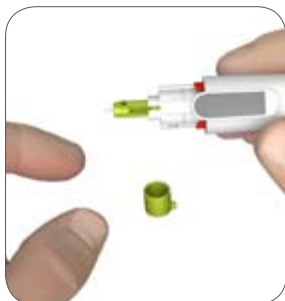

1. mylife™ Softlance lancing device and mylife™ Lancets multicolor after use.



2. To prevent puncture wounds the lancet can easily be pierced into the cap.



3. Hygienic and safe ejection of the lancet without touching it.





mylife™ Lancets high quality product range

Ideal for adults and children

mylife™ Lancets standard

High quality lancets for everyday use to ensure optimum blood flow. Designed to make blood sampling easier and more comfortable.

mylife™ Lancets multicolor

Recapable lancets for even safer disposal. Ideal for children. Available in seven different colors.



Maintains sterility for 5 years

Sterilizing dose determined in accordance with ISO 11137:1995

Ref. No.	Qty.	Description	Gauge	Needle Diameter	Point Length	Body Outer Diameter
7100335	200 pcs	mylife™ Lancets standard	30G	0.298 ~ 0.320 mm	3.3 +/- 0.25 mm	6.3 +/- 0.5 mm
7100336	200 pcs	mylife™ Lancets multicolor	30G	0.298 ~ 0.320 mm	3.3 +/- 0.25 mm	6.3 +/- 0.5 mm

More information on www.mylife-diabetescare.com

Ypsomed Distribution AG // Brunnmattstrasse 6 // 3401 Burgdorf
T +41 34 424 41 11 // F +41 34 424 41 22 // www.ypsomed.com